Instructions for Use Sterile Metal Bone Screw

Metal Bone Screws are used either to fixation plates onto bones or as big screws to hold bone fragments together. The screws are differentiated by the manner in which they are inserted into bones, their function, their size, and the type of bone they are intended for.

- Material: Metal Bone Screws are made of Titanium alloy.
Intended Purpose: Metal Bone Screw is intended to be used alone or in combination with bone plates for temporary fixation, correction and stabilization of limb fractures.
Indication: Metal Bone Screw is indicated for internal fixation of limb fractures.
Contraindications: (1) General or local infection, osteomyelitis. (2) Patient can't tolerate operation or anesthesia because of poor health.
Warnings and Precautions: (1) Preoperative planning is a requirement. The surgeons should choose the proper implants according to preoperative images and by using relating measuring instruments such as depth gauges and trials.

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Table with 2 columns: Manufacturer, Date of Manufacture, Batch code, Sterilized using irradiation, Authorized Representative, Keep away from sunlight, Consult instructions for use, Medical device.

Shanghai International Holding Corp. GmbH (Europa) Eifendstrasse 89, Hamburg, Germany

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Organismo notificado: TÜV SÜD Product Service GmbH